



**KALANI**  
finest natural home linen

## INSTRUCTIONS FOR WEARING AND CARING FOR YOUR 100% COTTON FABRIC MASK

- Wear this mask when you are in contact with people other than those you live with.
- Always check that the mask fits properly and covers your mouth and nose.
- Caution: If you are ill, this mask is not suitable. Ask your doctor for advice.

### HOW TO WEAR THE MASK?



#### BEFORE YOU PUT IT ON:

1. Before touching the mask, wash your hands with soap and water or a hydro-alcoholic solution.
2. Inspect the mask and make sure that there is no holes, tears or damage.
3. It is recommended to wear the mask on bare skin, avoiding contact with the hair.
4. Never modify the mask in any way whatever it is.



#### TO PUT IT ON:

1. Hold the mask by the straps.
2. Place it at the level of your mouth.
3. Put the upper straps behind your head and tie a knot. Repeat with the lower straps behind your neck.
4. Adjust the mask by unfolding it so that it covers your nose, mouth and chin.



#### WHEN YOU WEAR IT:

1. Avoid touching and moving it.
2. Never put it in a waiting position on the forehead or on your chin.

### HOW TO CLEAN THE MASK ?

#### BEFORE USE :

1. Store the Mask in a dry place away from direct sunlight. Store it in a maximum humidity of 75% and at a temperature between -2°C and +55°C in order to preserve all its characteristics.

#### FOR WASHING:

1. Machine wash the mask with detergent. Choose a wash programme that includes at least 30 min at 60°C minimum. You can put it in the washing machine with your laundry.
2. Don't forget to remove and discard any filter that may have been placed between the 2 layers of fabric before washing it.
3. After touching the mask, wash your hands with soap and water or hydroalcoholic solution.
4. When the machine is finished, take the mask out of the machine. and put it back in shape by reshaping the folds without forcing.



TAKE CARE OF YOURSELF  
AND OTHERS

## COVID-19

### WARNING

To protect your health and the health of others, it is very important to observe these operating instructions.

Despite all the care taken in the manufacturing of these masks, it is the sole responsibility of the user, and under no circumstances the manufacturer, or the supplier, to ensure that the mask is suitable for the characteristics of the environment in which it is to be used.

- This mask is not intended for use by nursing staff.
- This mask does not replace barrier gestures (regular hand washing, physical social distancing, reducing contact with other people). It adds a physical barrier, especially when you are in close contact with other people.



#### YOU HAVE TO CHANGE THE MASK:

1. When you wore the mask for four hours.
2. When you want to eat or drink.
3. When it becomes difficult to breathe.
4. If the mask gets wet.
5. If the mask is damaged.
6. If the mask no longer fits properly against your face.



#### TO REMOVE IT:

1. Wash your hands with soap and water, or hydro-alcoholic solution.
2. Unhook the lower and then the upper straps to remove the mask from your face.
3. While waiting to wash it, isolate it in a plastic bag or medical bag.
4. If the mask is worn out, throw it away immediately in a closable waste bin.
5. Finally: Wash your hands again with a water and soad or hydro-alcoholic solution.

#### AFTER WASHING:

1. Dry the mask completely. (inner layers must also be completely dry)
2. To do this, you can: use a tumble dryer (making sure to clean the filters of the dryer and wash your hands afterwards), use a hair dryer, or air dry your mask on a clean and disinfected surface.
3. Steam-iron your mask at the maximum temperature of your skin.
4. As soon as the mask is dry, store it in a clean, airtight package (e.g., plastic bag or freezer bag).

#### CONTACTEZ-NOUS SI VOUS AVEZ DES QUESTIONS

If you have any questions about our masks or these instructions for use, please contact us at the following address:

KALANI S.A.  
Rue de Rosières 2  
1332 Genval, Belgium  
Phone: +32(0)2 654 18 11  
Email: [info@kalani-home.com](mailto:info@kalani-home.com)  
Website: [www.kalani-home.com](http://www.kalani-home.com)